

**MAGELLAN®**

**Complete Disposable Kit**

**Explanation of symbols on package labeling**

	Lot Number
	Catalog Number
	Use by
	Quantity
	Do Not Reuse
	Fluid Path Sterilized Using Ethylene Oxide
	Fluid Path Sterilized Using Irradiation
	Fluid Path Sterilized Using Steam or Dry Heat
	Not made with natural rubber latex
	Nonpyrogenic
	United States Pharmacopeia
	This Way Up
	Manufacturer
	Fragile, Handle with Care
	Consult Instructions for Use
	Caution, Consult Accompanying Documents
	Caution: Federal law restricts this device to sale by or on the order of a (licensed healthcare practitioner).

**INTRODUCTION**

**Indications for Use**

The Arterioocyte Medical Systems, Inc. MAGELLAN® Complete Disposable Kit is intended for use only with the MAGELLAN® Autologous Platelet Separator Instrument. See Magellan® Platelet Separator Indications for Use.

**Contraindications**

The Use of the Arterioocyte Medical Systems, Inc. MAGELLAN® Disposables Kit is contraindicated for a hemodynamically unstable or hypercoagulable patient.

Use of this product for pediatric patients should be approached carefully. Withdrawing blood from pediatric patients should be at a physician's specific direction with attention given to avoiding any significant reduction in the patient's blood volume.

**Caution:** Medications that adversely affect a patient's coagulation system may inhibit the use of platelet separation system therapy.

**Warning:** Reprocessing may compromise the structural integrity of the device and/or lead to device failure. Reuse of this single patient use device creates a potential risk of patient or user infections. Contamination of the device may lead to injury, illness or death of the patient.

**Precautions**

1. Only Arterioocyte Medical Systems sterilized disposable kits are approved for patient use with the MAGELLAN® Platelet Separator Instrument. It is important that aseptic technique be used to minimize the possibility of contamination of the disposable components and/or patient.
2. Store all disposable components in a dry place away from extremes of environmental conditions.
3. Materials used in the Arterioocyte Medical Systems, Inc. MAGELLAN® Disposables Kit may be sensitive to chemicals (such as solvents and certain detergents). Under certain adverse conditions, exposure to these chemicals (including vapors) may cause the plastics to fail or malfunction. Visually inspect the contents of the disposable kit. Should any evidence of damage to the components be found during inspection or setup, do not

use the disposable components. Do not use silicone oils or greases near disposable components.

**System Description**

The Arterioocyte Medical Systems, Inc. MAGELLAN® Complete Disposable Kit consists of sterile fluid pathway single-patient-use components necessary for each platelet separation procedure with the MAGELLAN® Autologous Platelet Separator Instrument.

**Cautions**


1. Caution: Federal Law (USA) restricts this device to be sold by or on the order of physician.
2. Refer to the System Operator's manual supplied with the instrument before performing a platelet separation procedure using the components of this kit. Treat all blood and fluids using Universal Precautions for bloodborne pathogens.
3. Each platelet separation procedure using the Arterioocyte Medical Systems Inc. MAGELLAN® Platelet Separator Instrument requires one MAGELLAN® Platelet Separator Disposable Kit. The separation chamber and associated tubing can be used with the same patient for up to three complete separation cycles.
4. Use only Arterioocyte Medical Systems disposable accessories.
5. Inspect the kit prior to use. Do not use the kit if any component or the tray is damaged or opened.
6. Throughout procedure, make certain all tubing is free of any kinks, twists, or flat areas. All components of this kit are sterile fluid pathway single-patient-use. Do not use the ACD-A anticoagulant unless the solution is clear and the seal is intact. The ACD-A anticoagulant supplied in this kit is not for preparation of blood products for transfusion or for direct intravenous infusion. Discard the unused portion. Do not reuse.

**How Supplied**

Contents of the Arterioocyte Medical Systems, Inc. MAGELLAN® Complete Disposable Kit (see Figure 1):

- A. One (1) 50-mL bag ACD-A anticoagulant
- B. One (1) 60-mL syringe (Syringe 2)
- C. One (1) 10-mL syringe (Syringe 1)
- D. One (1) 5-mL syringe
- E. One (1) IV Site Prep Kit
- F. One (1) Separation chamber with attached tubing
- G. One (1) 18 gauge x 3.8 cm (1.5") needle
- H. One (1) phlebotomy needle set with luer-lock connector
- I. One (1) tubing clamp (For phlebotomy needle set)
- J. Four (4) syringe tip caps
- K. Instruction For Use

**Note:** If more than one separation cycle will be performed with the same patient, additional 10-mL and 60-mL <sup>1</sup>BD™ syringes are required and can be purchased from Arterioocyte Medical Systems.

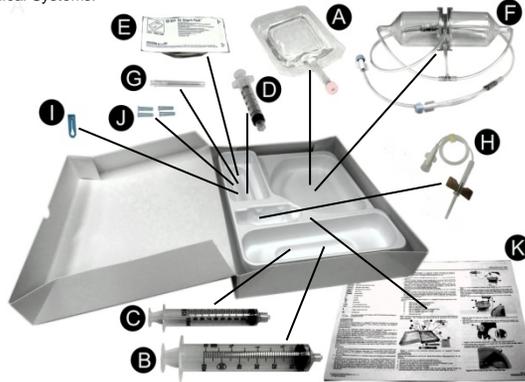


**Figure 1.** Contents of the Arterioocyte Medical Systems, Inc. MAGELLAN® Complete Disposable Kit.

**INSTRUCTIONS**

**Caution:** All components of this kit are sterile fluid pathway single-patient-use. Do not resterilize.

1. Remove the cover from the separator disposable kit tray.
2. Be certain that all components are present and undamaged.  
**Caution:** Do not use the kit if any component or the tray is damaged or open.
3. Remove the separation chamber package from the tray.
4. Peel open the lid on the chamber package.


5. Holding the platelet separation chamber with the vent facing upward, thread the attached tubing through the center of the chamber caddy (see Figure 2A and B).

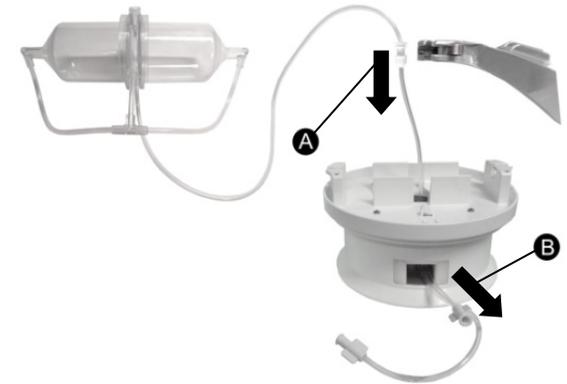

**Figure 2.** Thread the separation chamber tubing through the center of the chamber caddy.


6. Install the platelet separation chamber into the centrifuge caddy, making certain that both ends of the chamber are properly located in the caddy notches (see Figure 3A). Place T-connector in slot (see Figure 3B), and tubing under retainer on top surface of caddy (see Figure 3C). Press tubing down into groove on outer edge of chamber caddy (see Figure 3D).

**Caution:** Failure to install the separation chamber properly may result in error codes.

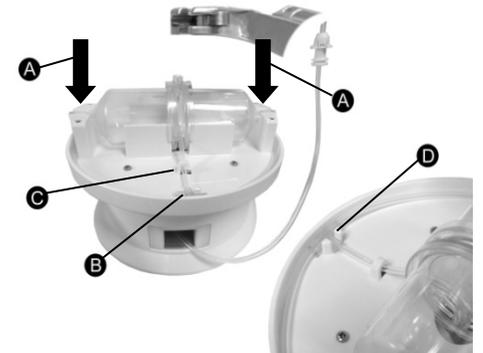

**Figure 3.** Install the separation chamber into the caddy and place tubing into slot and under retainer.


7. Rotate the tubing collar so that its shape aligns with the opening in the support arm. Slide the tubing collar into the support arm (see Figure 4A) and close the latch (see Figure 4B).

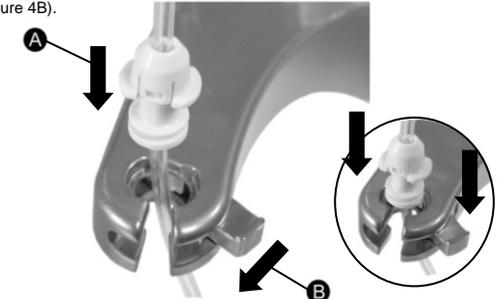

**Figure 4.** Attach chamber tubing collar to the instrument tubing support arm.


8. Press tubing down into groove on support arm and place tubing through notch in centrifuge ridge (see Figure 5).

**Caution:** Make certain all tubing is free of any kinks, twists, or flat areas.

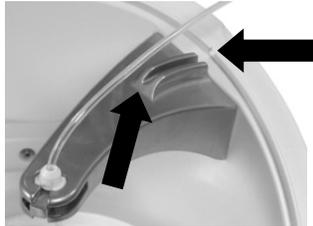

**Figure 5.** Place tubing through notch in centrifuge ridge.

- Prepare the syringe pumps on the front of the instrument by rotating the plunger drivers to the open position (see Figure 6A), while sliding them to the lowest points (see Figure 6B).

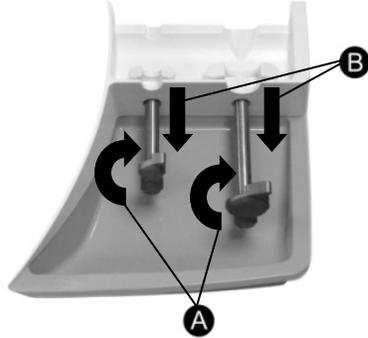

**Figure 6.** Within the syringe receptacles, slide the open plunger drivers to the lowest points.

- Prepare the patient for venipuncture according to standard clinical practice using the IV Site Prep Kit, if necessary.
- Using aseptic technique, draw the appropriate volume of anticoagulant from the ACD-A anticoagulant bag into syringe 2 (60 mL syringe) using the 18 gauge needle. Refer to Table 1 for the appropriate volumes of ACD-A and blood.

**Table 1. Volume of ACD-A and corresponding volumes of blood to achieve anticoagulated blood containing ~ 7 parts blood: 1 part ACD-A.**

Total Volume of Anticoagulated Blood (mL)	Volume of ACD-A (mL)	Volume of Blood Drawn (mL)
30	4.0	26.0
40	5.0	35.0
50	6.0	44.0
60	8.0	52.0

**Caution:** Do not use the ACD-A anticoagulant unless the solution is clear and the seal is intact. Do not reuse the ACD-A supplied in this kit except for multiple separation cycles with the same patient. Discard unused portion.

**Note:** Alternative methods can be used to collect patient blood. As appropriate, disregard references to the phlebotomy needle set.


- Hook the tubing clamp onto the tubing of the phlebotomy needle set. Do not clamp at this time.
- Attach the phlebotomy needle set with tubing clamp to the ACD-A primed 60 mL syringe (syringe 2).
- Prime the phlebotomy needle set by slowly pushing the plunger of syringe 2 until ACD-A reaches the top of the tubing closest to the needle.
- Slowly draw the appropriate volume of patient blood. Gently mix with ACD-A throughout the blood draw for thorough distribution. Refer to Table 1 for the appropriate volumes of ACD-A and blood.
- Once the appropriate amount of the patient's blood is drawn into syringe 2, close the tubing clamp prior to removing the phlebotomy needle set from the patient.

**Note:** Do not exceed 60 mL total volume.

**Note:** If multiple separation cycles are to be performed with the same patient, repeat steps 11, 15 and 16 to prepare and fill syringes.

- Disconnect syringe 2 from the IV tubing. Set aside the syringe and discard the needle and IV tubing utilizing appropriate local procedures.


- Remove the luer connector cap from the end of the shorter length of chamber tubing and attach syringe 1 (10 mL syringe, see Figure 7A).
  - Evacuate all air from syringe 1 into the chamber tubing line.
  - Place syringe 1 into the appropriate syringe pump receptacle.
  - Rotate and slide the plunger driver to engage the syringe 1 plunger (see Figure 7B).
  - Ensure whole blood is thoroughly mixed using gentle agitation prior to placing in the syringe pump receptacle.
  - Remove the luer connector cap from the longer length of chamber tubing and attach syringe 2 (see Figure 7C).
  - Evacuate all air from syringe 2 into the chamber tubing line.
- Note:** Failure to remove air from syringes may compromise the quality of the product.
- Place syringe 2 into the appropriate syringe pump receptacle.
  - Rotate and slide the plunger driver to engage the syringe 2 plunger (see Figure 7D).

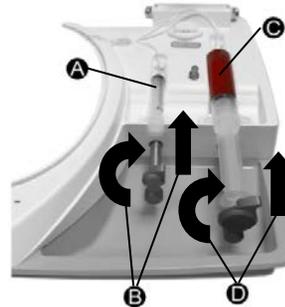

**Figure 7.** Attach tubing, load syringes, and engage syringe plunger drivers.

**Caution:** Make certain all tubing is free of any kinks, twists, or flat areas.

- Gently stretch the tubing attached to syringe 2 and slide it completely into pinch valve opening (see Figure 8).

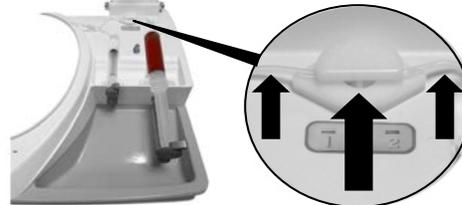

**Figure 8.** Stretch syringe 2 tubing and slide into pinch valve.

**ARTERIOCYTE MEDICAL SYSTEMS INC. DISPOSABLES KIT LIMITED WARRANTY<sup>2</sup> (U.S.)**

**THE FOLLOWING LIMITED WARRANTY APPLIES TO UNITED STATES CUSTOMERS ONLY:**

- This Limited Warranty provides the following assurance to the customer who receives the **Arterioocyte Medical Systems, Inc. MAGELLAN<sup>®</sup> Complete Disposable Kit**, hereafter referred to as the "Product":
  - Should the Product fail to function within normal tolerances due to a defect in materials or workmanship prior to its "Use Before Date", Arterioocyte Medical Systems will at its option: (a) issue a credit equal to the Purchase Price, as defined in Subsection A (2), against the purchase of the replacement Product or (b) provide a functionally comparable replacement Product at no charge.
  - As used herein, Purchase Price shall mean the lesser of the net invoiced price of the original, or current functionally comparable, or replacement Product.
- To qualify for the Limited Warranty, these conditions must be met:
  - The Product must be used prior to its "Use By" date.
  - The unused portion of the Product must be returned to Arterioocyte Medical Systems Inc. within 60 days after use and shall be the property of Arterioocyte Medical Systems.
  - The Product must not have been altered or subjected to misuse, abuse or accident.
  - The Product may not have been used by any other customer.
- This Limited Warranty is limited to its express terms. In particular:
  - Except as expressly provided by this Limited Warranty, **ARTERIOCYTE MEDICAL SYSTEMS IS NOT RESPONSIBLE FOR ANY DIRECT, INCIDENTAL OR CONSEQUENTIAL DAMAGES BASED ON ANY DEFECT, FAILURE OR MALFUNCTION OF THE PRODUCT, WHETHER THE CLAIM IS BASED ON WARRANTY, CONTRACT, TORT OR OTHERWISE.**

- This Limited Warranty is made only to the customer in whom the Product was used. AS TO ALL OTHERS, ARTERIOCYTE MEDICAL SYSTEMS MAKES NO WARRANTY, EXPRESS OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OF MERCHANTABILITY, OR FITNESS FOR A PARTICULAR PURPOSE WHETHER ARISING FROM STATUTE, COMMON LAW, CUSTOM OR OTHERWISE. NO EXPRESS OR IMPLIED WARRANTY TO THE CUSTOMER SHALL EXTEND BEYOND THE PERIOD SPECIFIED IN A(1) ABOVE. THIS LIMITED WARRANTY SHALL BE THE EXCLUSIVE REMEDY AVAILABLE TO ANY PERSON.
- The exclusions and limitations set out above are not intended to, and should not be construed so as to contravene mandatory provisions of applicable law. If any part or term of this Limited Warranty is held to be illegal, unenforceable or in conflict with applicable law by a court of competent jurisdiction, the validity of the remaining portions of the Limited Warranty shall not be affected, and all rights and obligations shall be construed and enforced as if this Limited Warranty did not contain the particular part or term held to be invalid. This Limited Warranty gives the customer specific legal rights. The customer may also have other rights which vary from state to state.

No person has any authority to bind Arterioocyte Medical Systems to any representation, condition or warranty except this Limited Warranty.

**DISPOSABLES LIMITED WARRANTY – ARTERIOCYTE MEDICAL SYSTEMS, INC. (OUTSIDE U.S.)**

**THE FOLLOWING LIMITED WARRANTY APPLIES TO CUSTOMERS OUTSIDE THE UNITED STATES.**

- This LIMITED WARRANTY provides assurance for the customer who receives an Arterioocyte Medical Systems<sup>®</sup> ("AMS") **MAGELLAN<sup>®</sup> Complete Disposable Kit** "Product", that should the Product fail to function to specification, AMS will issue a credit equal to the original Product purchase price (but not to exceed the value of the replacement Product) against the purchase of any AMS replacement Product used for that customer. THE WARNINGS CONTAINED IN THE PRODUCT LABELLING ARE CONSIDERED AN INTEGRAL PART OF THIS LIMITED WARRANTY. CONTACT YOUR LOCAL AMS REPRESENTATIVE TO OBTAIN INFORMATION ON HOW TO PROCESS A CLAIM UNDER THIS WARRANTY.
- To qualify for the LIMITED WARRANTY, these conditions must be met.
  - The Product must be used prior to its "Use By" date.
  - The Product must be returned to AMS within sixty (60) days after use and shall be the property of AMS.
  - The Product must not have been altered or subjected to misuse, abuse or accident.
  - The Product must not have been used more than one time for any customer.
  - The Product must be used in conformity with the Product, of which this LIMITED WARRANTY is an integral part.
- This LIMITED WARRANTY is limited to its express terms. In particular:
  - In no event shall any replacement credit be granted where there is evidence of improper handling, improper use or material alteration of the replaced Product.
  - AMS is not responsible for any incidental or consequential damages based on any use, defect or failure of the Product, whether the claim is based on warranty, contract, tort, patent infringement or otherwise.
- This LIMITED WARRANTY does not cover those parts that, by their very nature, are likely to deteriorate or which AMS considers should be periodically replaced consistently with normal routine or preventative maintenance requirements.
- The exclusions and limitations set out above are not intended to, and should not be construed so as to, contravene mandatory provisions of applicable law. If any part or term of this LIMITED WARRANTY is held by any court of competent jurisdiction to be illegal, unenforceable, or in conflict with applicable law, the validity of the remaining portion of the LIMITED WARRANTY shall not be affected, and all rights and obligations shall be construed and enforced as if this LIMITED WARRANTY did not contain the particular part or term held to be invalid.
- No representative, agent, dealer, retailer, or intermediary of AMS shall have authorization to amend the contents of this LIMITED WARRANTY.
- The validity, interpretation and performance of the agreement for which this LIMITED WARRANTY is issued, as well as any disputes relating or connected thereto is controlled by and construed under the laws of the State of Delaware, USA.

**Manufacturer:**

Arterioocyte Medical Systems, Inc.  
45 South St. Hopkinton, MA 01748 USA - Internet: [www.arterioocyte.com](http://www.arterioocyte.com)  
Toll-free USA: 1-866-660-AMSI (2674) Fax: 1-508 -497-8951

<sup>1</sup> BD<sup>™</sup> is a trademark of Becton, Dickinson and Company.

<sup>2</sup> This Limited Warranty is provided by Arterioocyte Medical Systems, Inc., 45 South St., Hopkinton, MA 01748 USA. It applies only in the United States. Areas outside the United States should contact their local Arterioocyte Medical Systems representative for exact terms of the Limited Warranty.


AMS Confidential

4 x 6.5 inches (101.6 mm x 165.1 mm)

**ARTERIOCYTE**  
*CELLULAR THERAPIES  
MEDICAL SYSTEMS*

---

**MAGELLAN<sup>®</sup>**  
Ratio Dispenser Kit

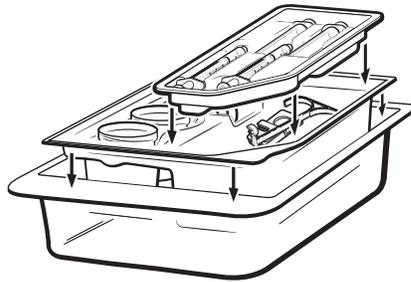




Instructions For Use

---


AMS95125LBL Rev. 2

**AMS Confidential**

4 x 6.5 inches (101.6 mm x 165.1 mm)

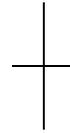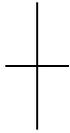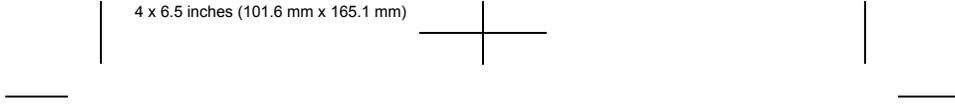

AMS95125LBL Rev. 2

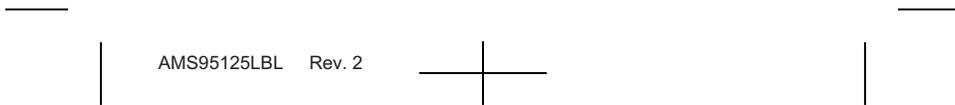

4 x 6.5 inches (101.6 mm x 165.1 mm)

**Explanation of symbols on package labeling**


---

	Caution, Consult Accompanying Documents
	Lot Number
	Catalog Number
	Sterilized Using Ethylene Oxide
	Latex Free
	Nonpyrogenic
	Do Not Reuse
	Use By
	Open Here
	Fragile, Handle with Care
	This Way Up
	Quantity
	For US Audiences Only

---

4 x 6.5 inches (101.6 mm x 165.1 mm)

**AMS Confidential**

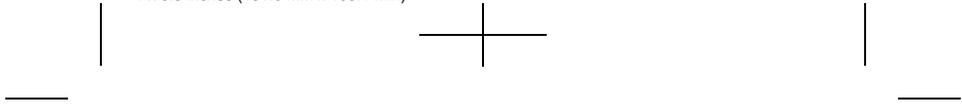

2

AMS95125LBL Rev. 2



4 x 6.5 inches (101.6 mm x 165.1 mm)

## INTRODUCTION

The Arterioocyte Medical Systems, Inc. Magellan® Ratio Dispenser Kit consists of single use components to be used in conjunction with a Arterioocyte Medical Systems, Inc. Magellan® Dispenser Tip.

### Intended use

The components of this Arterioocyte Medical Systems, Inc. Magellan® Ratio Dispenser Kit, when used with a tip, are intended to assist the user in simultaneously delivering two non-homogenous liquids to the same treatment area(s).

The Magellan® Ratio Dispenser Kit is intended for the application of fluids, as deemed necessary by the surgeon's determination of the clinical use requirements, to facilitate the preparation of soft tissue prior to repair.

**Warning:** Reprocessing may compromise the structural integrity of the device and/or lead to device failure. Reuse of this single patient use device creates a potential risk of patient or user infections. Contamination of the device may lead to injury, illness or death of the patient.

### How Supplied

All components are shipped sterile.

The Magellan® Ratio Dispenser Kit consists of the following components:

- One (1) procedure tray
- Three (3) 60-mL sample cups with lids
- Two (2) 12-mL syringes
- Two (2) 1-mL syringes
- Two (2) dispenser handles
- Two (2) dual syringe plunger clips

### Accessories

One (1) Magellan® Dispenser Tip is required for the use of this dispenser kit.

Dispenser Tips are available from Arterioocyte Medical Systems.

### Cautions

**USA** Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

Kit components are for single use only. Do not resterilize.

## INSTRUCTIONS

1. Using aseptic technique, remove the components of the dispenser kit from the sterile package and a dispenser tip from its sterile package.
2. Fill the larger 12-mL syringe with liquid.
3. Fill the smaller 1-mL syringe with a second liquid to approximately the same level.

**AMS Confidential**

4 x 6.5 inches (101.6 mm x 165.1 mm)

4. Snap the dispenser handle to the 12mL syringe (see Figure 1, number 1).
5. Attach each syringe to the dispenser tip (see Figure 1, numbers 2 and 3).
6. Snap the plunger clip to both syringe plungers (see Figure 1, number 4).  
Figure 2 shows the assembled dispenser.
7. Depress the syringe plunger clip to dispense the desired volume of the two liquids.

4 English Instructions for Use

AMS95125LBL Rev. 2

4 x 6.5 inches (101.6 mm x 165.1 mm)

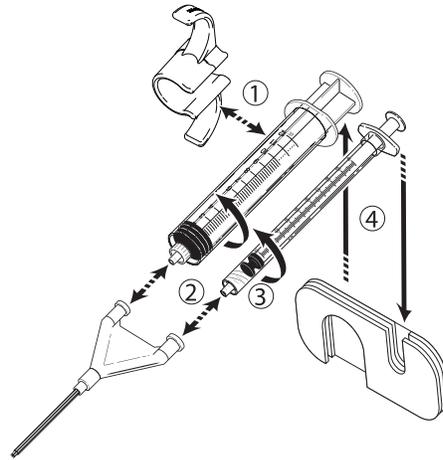


Figure 1.

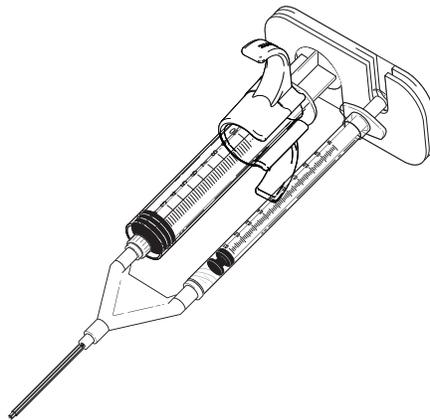


Figure 2.

4 x 6.5 inches (101.6 mm x 165.1 mm)

**ARTERIOCYTE MEDICAL SYSTEMS INC. DISPOSABLES  
KIT LIMITED WARRANTY<sup>1</sup> (U.S.)**

**THE FOLLOWING LIMITED WARRANTY APPLIES TO UNITED STATES  
CUSTOMERS ONLY:**

- A. This Limited Warranty provides the following assurance to the customer who receives the **Arterioocyte Medical Systems, Inc. Magellan<sup>®</sup>Ratio Dispenser Kit**, hereafter referred to as the "Product":
- (1) Should the Product fail to function within normal tolerances due to a defect in materials or workmanship prior to its "Use Before Date", Arterioocyte Medical Systems will at its option: (a) issue a credit equal to the Purchase Price, as defined in Subsection A(2), against the purchase of the replacement Product or (b) provide a functionally comparable replacement Product at no charge.
  - (2) As used herein, Purchase Price shall mean the lesser of the net invoiced price of the original, or current functionally comparable, or replacement Product.
- B. To qualify for the Limited Warranty, these conditions must be met:
- (1) The Product must be used prior to its "Use By" date.
  - (2) The unused portion of the Product must be returned to Arterioocyte Medical Systems and within 60 days after use and shall be the property of Arterioocyte Medical Systems.
  - (3) The Product must not have been altered or subjected to misuse, abuse or accident.
  - (4) The Product may not have been used by any other customer.
- C. This Limited Warranty is limited to its express terms. In particular:
- (1) Except as expressly provided by this Limited Warranty, ARTERIOCYTE MEDICAL SYSTEMS IS NOT RESPONSIBLE FOR ANY DIRECT, INCIDENTAL OR CONSEQUENTIAL DAMAGES BASED ON ANY DEFECT, FAILURE OR MALFUNCTION OF THE PRODUCT, WHETHER THE CLAIM IS BASED ON WARRANTY, CONTRACT, TORT OR OTHERWISE.
  - (2) This Limited Warranty is made only to the customer in whom the Product was used. AS TO ALL OTHERS, ARTERIOCYTE MEDICAL SYSTEMS MAKES NO WARRANTY, EXPRESS OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OF MERCHANTABILITY, OR FITNESS FOR A PARTICULAR PURPOSE WHETHER ARISING FROM STATUTE, COMMON LAW, CUSTOM OR OTHERWISE. NO EXPRESS OR IMPLIED WARRANTY TO THE CUSTOMER SHALL EXTEND BEYOND THE PERIOD SPECIFIED IN A(1) ABOVE. THIS LIMITED WARRANTY SHALL BE THE EXCLUSIVE REMEDY AVAILABLE TO ANY PERSON.
  - (3) The exclusions and limitations set out above are not intended to, and should not be construed so as to contravene mandatory provisions of

<sup>1</sup> This Limited Warranty is provided by Arterioocyte Medical Systems, Inc., 45 South Street, Hopkinton, MA 01748 USA. It applies only in the United States. Areas outside the United States should contact their local Arterioocyte Medical Systems representative for exact terms of the Limited Warranty.

4 x 6.5 inches (101.6 mm x 165.1 mm)

applicable law. If any part or term of this Limited Warranty is held to be illegal, unenforceable or in conflict with applicable law by a court of competent jurisdiction, the validity of the remaining portions of the Limited Warranty shall not be affected, and all rights and obligations shall be construed and enforced as if this Limited Warranty did not contain the particular part or term held to be invalid. This Limited Warranty gives the customer specific legal rights. The customer may also have other rights which vary from state to state.

- (4) No person has any authority to bind Arterioocyte Medical Systems to any representation, condition or warranty except this Limited Warranty.

4 x 6.5 inches (101.6 mm x 165.1 mm)




**DISPOSABLES LIMITED WARRANTY – ARTERIOCYTE MEDICAL SYSTEMS, INC. (OUTSIDE U.S.)**

THE FOLLOWING **LIMITED WARRANTY** APPLIES TO CUSTOMERS OUTSIDE THE UNITED STATES.

- A. This LIMITED WARRANTY provides assurance for the customer who receives an Arterioocyte Medical Systems ® ("AMS") **Magellan® Ratio Dispenser Kit** "Product", that should the Product fail to function to specification, AMS will issue a credit equal to the original Product purchase price (but not to exceed the value of the replacement Product) against the purchase of any AMS replacement Product used for that customer. THE WARNINGS CONTAINED IN THE PRODUCT LABELLING ARE CONSIDERED AN INTEGRAL PART OF THIS LIMITED WARRANTY. CONTACT YOUR LOCAL AMS REPRESENTATIVE TO OBTAIN INFORMATION ON HOW TO PROCESS A CLAIM UNDER THIS WARRANTY.
- B. To qualify for the LIMITED WARRANTY, these conditions must be met:
- (1) The Product must be used prior to its "Use By" date.
  - (2) The Product must be returned to AMS within sixty (60) days after use and shall be the property of AMS.
  - (3) The Product must not have been altered or subjected to misuse, abuse or accident.
  - (4) The Product must not have been used more than one time for any customer.
  - (5) The Product must be used in conformity with the user instructions in the "User Manual" delivered to the customer together with the Product, of which this LIMITED WARRANTY is an integral part.
- C. This LIMITED WARRANTY is limited to its express terms. In particular:
- (1) In no event shall any replacement credit be granted where there is evidence of improper handling, improper use or material alteration of the replaced Product.
  - (2) AMS is not responsible for any incidental or consequential damages based on any use, defect or failure of the Product, whether the claim is based on warranty, contract, tort, patent infringement or otherwise.
- D. This LIMITED WARRANTY does not cover those parts that, by their very nature, are likely to deteriorate or which AMS considers should be periodically replaced consistently with normal routine or preventative maintenance requirements.
- E. The exclusions and limitations set out above are not intended to, and should not be construed so as to, contravene mandatory provisions of applicable law. If any part to term of this LIMITED WARRANTY is held by any court of competent jurisdiction to be illegal, unenforceable, or in conflict with applicable law, the validity of the remaining portion of the LIMITED WARRANTY shall not be affected, and all rights and obligations shall be construed and enforced as if this LIMITED WARRANTY did not contain the particular part or term held to be invalid.
- F. No representative, agent, dealer, retailer, or intermediary of AMS shall have authorization to amend the contents of this LIMITED WARRANTY.
- G. The validity, interpretation and performance of the agreement for which this LIMITED WARRANTY is issued, as well as any disputes relating or connected thereto is controlled by and construed under the laws of the State of Ohio, USA.

**AMS Confidential**

4 x 6.5 inches (101.6 mm x 165.1 mm)

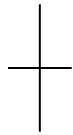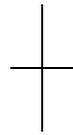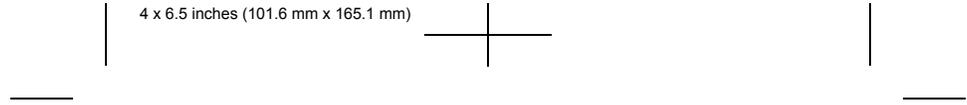

AMS95125LBL Rev. 2



AMS Confidential

4 x 6.5 inches (101.6 mm x 165.1 mm)

**ARTERIOCYTE**  
*CELLULAR THERAPIES*  
*MEDICAL SYSTEMS*

**Manufacturer:**

Arterioocyte Medical Systems, Inc.  
45 South St.  
Hopkinton, MA 01748  
USA  
Internet: [www.arterioocyte.com](http://www.arterioocyte.com)  
Toll-free USA: 1-866-660-AMSI(2674)  
Fax: 1-508-497-8951

© Arterioocyte Medical Systems, Inc. 2011  
All Rights Reserved  
AMS95125LBL Rev. 2

AMS95125LBL Rev. 2

AMS Confidential

**ARTERIOCYTE**  
Helping Patients Heal Faster™

**MAGELLAN®**  
Aspiration Kit

Instructions For Use

---

AMS Confidential

— |

+

| —

+

+

— |

+

| —

**Explanation of symbols on package labeling**

---

 Caution, Consult Accompanying Documents

 Lot Number

 Catalog Number

 Sterilized Using Ethylene Oxide

 Latex Free

 Nonpyrogenic

 Do Not Reuse

 Use by

 Fragile, Handle with Care

 This Way Up

 Quantity

 Phthalate

 Open Here

 For US Audiences Only

---

## INTRODUCTION

The Arteriocyte Medical Systems, Inc.' MAGELLAN® Aspiration Kit is a sterile convenience kit for aspirating bone marrow.

### Intended Use:

The Arteriocyte Medical Systems, Inc. MAGELLAN® Aspiration Kit is intended for use only with the MAGELLAN® Autologous Platelet Separator Instrument.

**Note:** Refer to the MAGELLAN® Autologous Platelet Separator Operator's Manual for Usage Instructions, Disclaimers, Warnings, Precautions, and Contraindications.

### Cautions


- Federal Law (USA) restricts this device to sale by or on the order of a physician.
- Kit components for single patient use only. Do not resterilize
- Refer to the System Operator's Manual supplied with the instrument, as well as Instruction for Use for components from other manufacturers inside this convenience kit before performing the procedure. Treat all blood, tissue and fluids using Universal Precautions for bloodborne pathogens.
- Use only Arteriocyte Medical Systems disposable accessories.
- Inspect the kit prior to use. Do not use the kit if any component of the tray is damaged or opened.

### Precautions

1. Only Arteriocyte Medical Systems, Inc. sterilized disposable kits are approved for patient use with the MAGELLAN® Autologous Platelet Separator Instrument. It is important that aseptic technique be used to minimize the possibility of contamination of the disposable components and/or patient.
2. Store all disposable components in a dry place away from extremes of environmental conditions.
3. Materials used in the Arteriocyte Medical Systems, Inc. MAGELLEN® Disposable Kit may be sensitive to chemicals (such as solvents and certain detergents).
4. Under certain adverse conditions, exposure to these chemicals (including vapors) may cause the plastics to fail or malfunction. Visually inspect the contents of the disposable kit. Should any evidence of damage to the components be found during inspection or setup, do not use the disposable components. Do not use silicone oils or greases near disposable components.


### How Supplied:

**AMS Confidential**

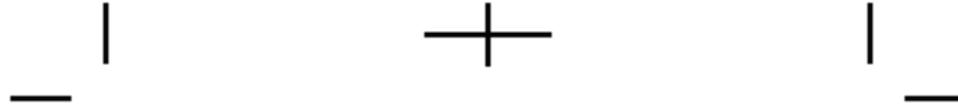

The MAGELLAN® Aspiration Kit consists of the following components:

- Two (2) 30 mL Syringes
- One (1) 60 mL Syringe
- One (1) 200 Micron Blood Component Filter
- One (1) Bone Marrow Aspiration Needle
- One (1) 3-Way Stopcock
- One (1) 18-G Needle
- Four (4) Syringe Tip Caps

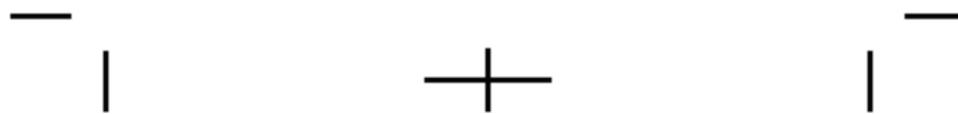

**200 Micron Blood Component Filter and Syringe Adapter  
Instructions for use**

Prime heparin flushed filter and syringes. Residual priming volume of saline is 6 mL. Maximum volume of product is 300 mL.

**Directions:**

Use aseptic technique. Single patient use.

1. Close the pinch clamp on the distal end of filter tubing.
2. While holding adaptor upright, squeeze the filter chamber and attach the syringe containing the blood component to the proximal end.
3. Release hold on chamber allowing blood components to enter filter chamber. Release pinch clamp and allow blood components to flow into distal tubing just to luer connector to de-air tubing and close pinch clamp.
4. Attach the collection syringe to distal tubing luer connector of the filter and open pinch clamp.
5. Gently pull back on the plunger of the collection syringe until resistance is met, or until the desired volume is reached. If resistance is met before desired volume is reached, disconnect proximal syringe and pull remaining blood components into collection syringe. Repeat if more than one syringe of blood component is drawn.
6. Disconnect the filtered blood component syringe from filter tubing and attach a sterile syringe cap.

Follow AABB guidelines.

**Manufacturer:**

Arteriocyte Medical Systems, Inc.  
45 South Street  
Hopkinton, MA 01748  
USA  
Internet: [www.arteriocyte.com](http://www.arteriocyte.com)  
Toll-free: 1-866-660-AMSI (2674)  
Fax: 508-497-8951

**Instructions for use of this Bone Marrow Aspiration Needle**

**A. INTENDED USE**

The Bone Marrow Aspiration Needle is intended for use in aspirating bone marrow specimen.

**B. CONTRAINDICATIONS**

For use only for bone marrow aspiration as determined by a licensed physician. The device is intended to be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of bone marrow aspiration. The procedure should be performed on patients that are suitable for such procedure only.

**Caution:** For single patient use only. Do not attempt to clean or resterilize this product. The design of this device may not perform as intended by the manufacturer if it is reused. The manufacturer cannot guarantee the performance, safety and reliability of a reprocessed device.


After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations. Carefully place the used needle in a sharps biohazard container after the procedure is completed.

**Note:** These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

**C. DIRECTIONS FOR USE**

1. Inspect the package for proper integrity. If undamaged, open the package using aseptic technique.
2. Remove the needle from the pouch and inspect the cutting edge of both cannula and stylet for any damage or other imperfections that would prevent the proper operation of the needle.
3. Following proper placing technique, locate the needle adjacent to the biopsy site.
4. Using gentle (but firm) pressure, advance the needle by rotating in an alternating clockwise-counterclockwise motion. Entrance into the marrow cavity is generally detected by decreased resistance.

**AMS Confidential**

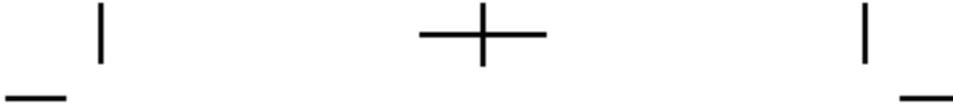

5. Rotate the stylet cap counterclockwise, and pull the stylet out of the cannula.

6. Attach a luer syringe to the cannula hub.


**Note:** Avoid using glass syringes because they may result in an inadequate seal and may break and fragment.

7. Apply negative pressure by quickly withdrawing the syringe plunger. Disengage the syringe and remove the aspirated sample.

**Caution:** Federal (USA) law restricts this device to sale by or on the order of a physician.



Manufactured by:  
Ranfac Corp.  
30 Doherty Ave.  
Avon, MA 02322 USA

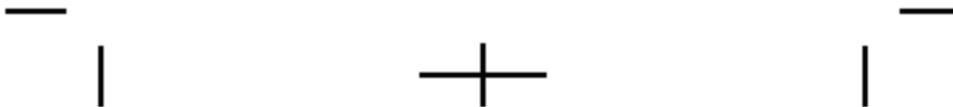

**ARTERIOCYTE MEDICAL SYSTEMS, INC. DISPOSABLES KIT  
LIMITED WARRANTY<sup>1</sup> (U.S.)  
THE FOLLOWING LIMITED WARRANTY APPLIES TO UNITED  
STATES CUSTOMERS ONLY:**

- A. This Limited Warranty provides the following assurance to the customer who receives the Arterioocyte Medical Systems, Inc. MAGELLAN<sup>®</sup> Aspiration Kit, hereafter referred to as the "Product":
- (1) Should the Product fail to function within normal tolerances due to a defect in materials or workmanship prior to its "Use Before Date", Arterioocyte Medical Systems will at its option:
    - (a) issue a credit equal to the Purchase Price, as defined in Subsection A (2), against the purchase of the replacement Product or
    - (b) provide a functionally comparable replacement Product at no charge.
  - (2) As used herein, Purchase Price shall mean the lesser of the net invoiced price of the original, or current functionally comparable, or replacement Product.
- B. To qualify for the Limited Warranty, these conditions must be met:
- (1) The Product must be used prior to its "Use By" date.
  - (2) The unused portion of the Product must be returned to Arterioocyte Medical Systems and within 60 days after use and shall be the property of Arterioocyte Medical Systems.
  - (3) The Product must not have been altered or subjected to misuse, abuse or accident.
  - (4) The Product may not have been used by any other customer.
- C. This Limited Warranty is limited to its express terms. In particular:
- (1) Except as expressly provided by this Limited Warranty, ARTERIOCYTE MEDICAL SYSTEMS IS NOT RESPONSIBLE FOR ANY DIRECT, INCIDENTAL OR CONSEQUENTIAL DAMAGES BASED ON ANY DEFECT, FAILURE OR MALFUNCTION OF THE PRODUCT, WHETHER THE CLAIM IS BASED ON WARRANTY, CONTRACT, TORT OR OTHERWISE.
  - (2) This Limited Warranty is made only to the customer in




whom the Product was used. AS TO ALL OTHERS, ARTERIOCYTE MEDICAL SYSTEMS MAKES NO WARRANTY, EXPRESS OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OF MERCHANTABILITY, OR FITNESS FOR A PARTICULAR PURPOSE WHETHER ARISING FROM STATUTE, COMMON LAW, CUSTOM OR OTHERWISE. NO EXPRESS OR IMPLIED WARRANTY TO THE CUSTOMER SHALL EXTEND BEYOND THE PERIOD SPECIFIED IN A(1) ABOVE. THIS LIMITED WARRANTY SHALL BE THE EXCLUSIVE REMEDY AVAILABLE TO ANY PERSON.

(3) The exclusions and limitations set out above are not intended to, and should not be construed so as to contravene mandatory provisions of applicable law. If any part or term of this Limited Warranty is held to be illegal, unenforceable or in conflict with applicable law by a court of competent jurisdiction, the validity of the remaining portions of the Limited Warranty shall not be affected, and all rights and obligations shall be construed and enforced as if this Limited Warranty did not contain the particular part or term held to be invalid. This Limited Warranty gives the customer specific legal rights. The customer may also have other rights which vary from state to state.

(4) No person has any authority to bind Arterioocyte Medical Systems to any representation, condition or warranty except this Limited Warranty.

<sup>1</sup> This Limited Warranty is provided by Arterioocyte Medical Systems, Inc., 45 South Street Hopkinton, MA 01748. It applies only in the United States. Areas outside the United States should contact their local Arterioocyte Medical Systems representative for exact terms of the Limited Warranty.

APES Confidential

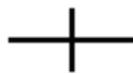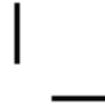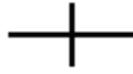

AMS Confidential

**ARTERIOCYTE**  
Helping Patients Heal Faster™

**Manufacturer:**

Arteriocyte Medical Systems, Inc.  
45 South Street  
Hopkinton, MA 01748  
USA  
Internet: [www.arteriocyte.com](http://www.arteriocyte.com)  
Toll-free: 1-866-660-AMSI (2674)  
Fax: 508-497-8951

©Arteriocyte Medical Systems, Inc.  
All Rights Reserved  
AMS95140LBL Rev 2