

PEEPACK





When you're caught short...

The **PeePack** is for men, women and kids. We all need to wee at some point in the day. But what if there are no public loos? And that's more and more the case, whether you're in the car, in town or out in the countryside with the family...

It's time for the PeePack!

Small, light, handy and ultra-practical. Easily fits in your handbag, work bag, rucksack, glove-box etc. **You'll always have a toilet handy for those moments of need**.

What's in the pack?



Applications







Golfers - 10th tee 'occasions'

Shift workers who can't easily leave their work station - e.g. crane drivers, lorry drivers

Care industry - avoid bed pans and urinal bottles with a disposable solution

Sporting types: sailing, hot-air ballooning, jogging, orienteering, hiking, camping, pilots...

Outdoor music festivals, carnivals and events
- avoid queues and unhygienic loos - it happens

Guaranteed solution for wheelchair users
- freedom to get out and about

Emergency services - chances are, there are no public loos at the scene of the incident

Contact us for more information.

Freephone: 0800 334 5742

Website: www.peepack.co.uk



PEEPACK

Science behind the magic

Leak-proof and hygienic, the **Pee Pack** comes in a blister pack weighing just 55g, which consists of a plastic pouch, a foldable funnel for female users, a cleansing wipe and user instructions.

Up to **750ml (1 ½ pints)** of urine are bound in a few seconds by the super-absorber crystals within the plastic pouch into an odour-resistant, leak-proof gel. Once 'business' is finished, the pouch (and funnel) can be sealed and disposed of in the household waste bin.

Each pack contains:

- 1 x crystal filled pouch
- 1 x funnel
- 1 x hand wipe

Pack size: 17cm L x 11.5cm W x 1cm D

Weight: 55g

How does it work?

- Open pack, remove sticker and unfold blue pouch.
- Men: Hold blue pouch below ring and 'relieve'.
- Ladies: Open cardboard funnel, insert pointed end into blue pouch, wide end against yourself, hold pouch below ring and 'relieve'. Fold up cardboard funnel and push all the way into pouch.
- **All:** Fold up pouch, secure with sticker and dispose into a suitable waste bin.























