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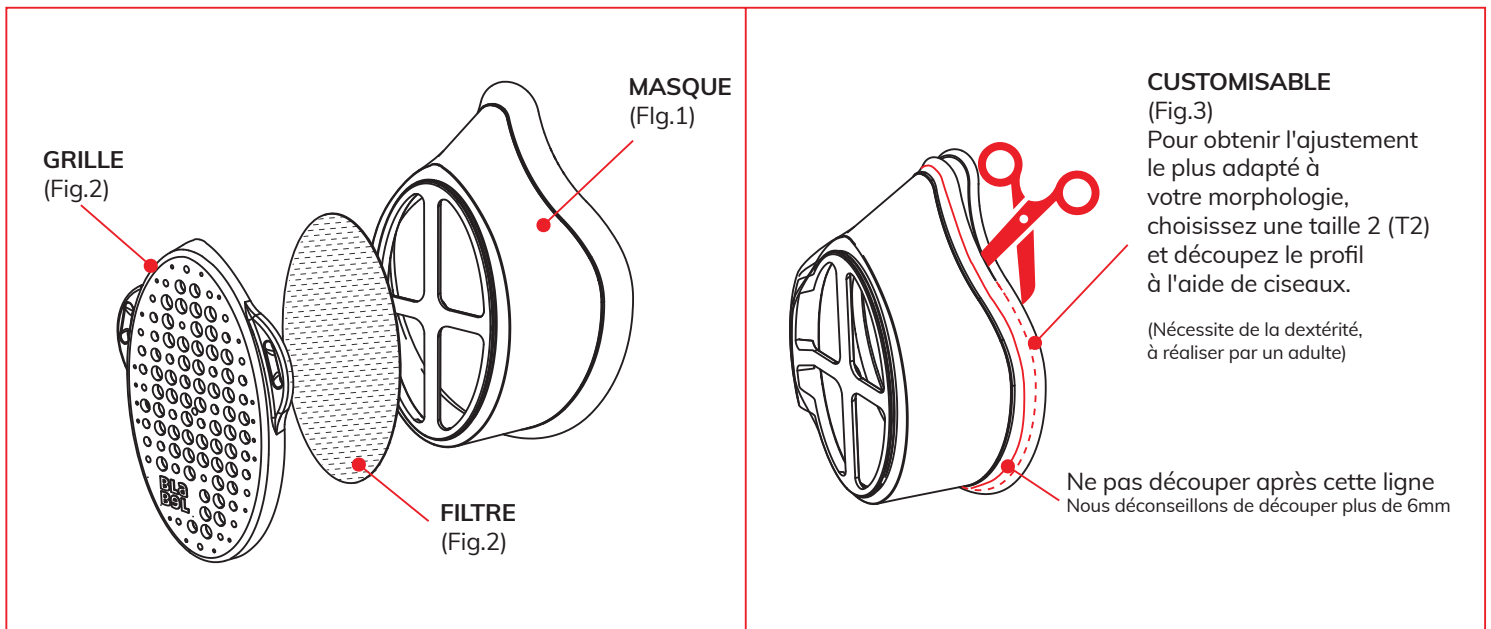
# EN/MSQ01 - FACE MASK

## MASK (Fig.1) AND GRID (Fig.2)

MSQ01 reduces the risk of infecting other people by droplet infection, in a similar way to a 3 ply mask. MSQ01 does not provide self-protection for the wearer.

## FILTER (Fig.3)

- MSQ01 comes with 5 filters (covid-19 valid protection certified by Centexbel filters - 88% filtration rate)
- Every filter can be washed up to 125 times (Do not wash alone, 80°C max)
- A clean filter must be inserted before each use.
- Please note that various materials may be used as filter.
- A cut out template is included in order to cut filters out of other materials.
- MSQ01 must never be used without a filter.
- Follow public health and social measures, including social distancing, while wearing MSQ01
- When wearing MSQ01, make sure that both nose and mouth are completely covered.

## CARE

Mask : MSQ01 should be thoroughly and regularly cleaned. Hand wash with hot water and detergent or in a dishwasher (60°C max). The filter should be removed before cleaning.

Filter : Washable up to 125 times at a temperature of 80°C max (Do not wash alone). Do not dry.

## SIZE

T1: Head circumference from 54 cm to 58 cm

T2: Head circumference from 59 cm to 65 cm

To get the best fit for your face, choose a size 2 (T2) and cut out the profile using scissors. (Fig. 3).

Please refer to [www.blabel.be](http://www.blabel.be) for more information on the appropriate use of MSQ01

**100% RECYCLED:** Made of Belgium sourced recycled plastics (Bio compatibility approved).

**100% LOCAL:** Invented, produced and assembled in Brussels.

**100% RECYCLABLE:** Fully recyclable.

**100% DURABLE:** Wash it, wash the filters and (re-)use again. And again.

**100% ERGONOMIC:** Custom fit, no fog on your glasses.

## COMPOSITION

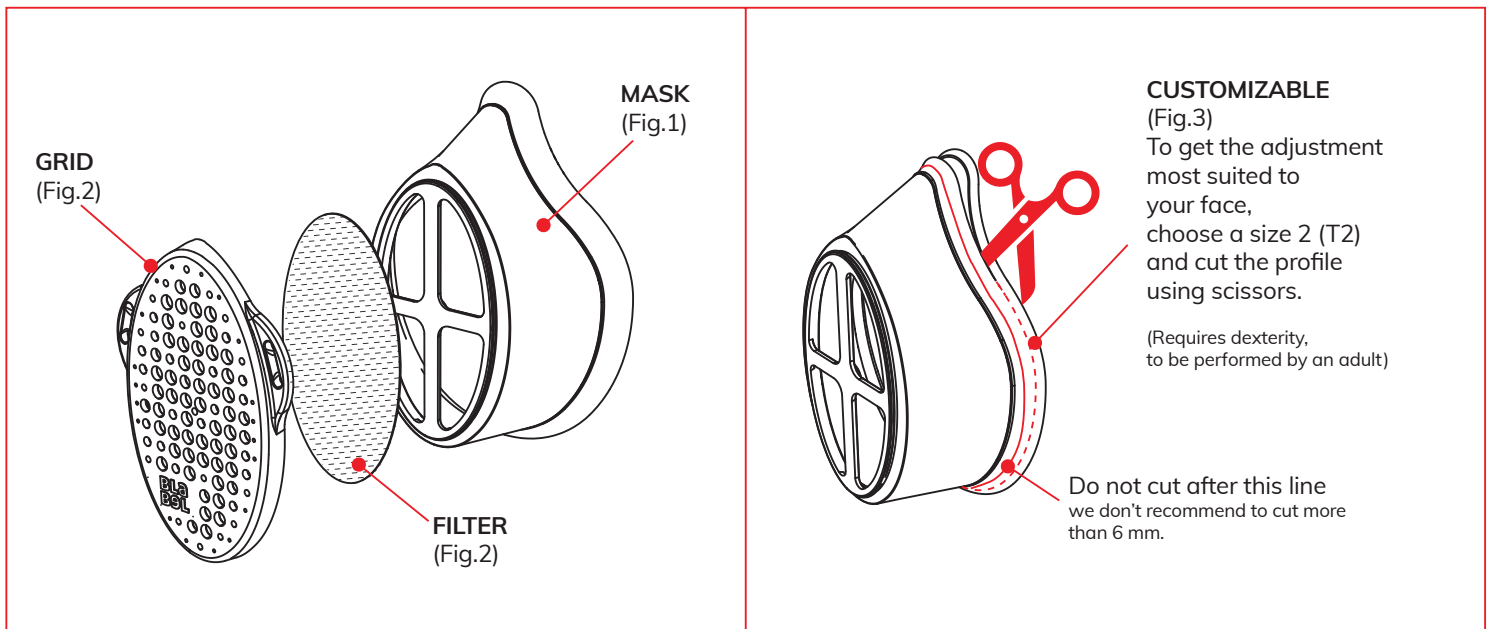
-Mask: LDPE

-Filter: 100% Polyester

-Elastic: 75% Polyester - 25% Lycra

## Legal notice:

MSQ01 does not provide self-protection for the wearer and can not be used as a medical device.

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